

# CytoSorb®



## Best practice flowchart antithrombotic removal



### Patient selection

Patients undergoing cardiac surgery who were pretreated with ticagrelor and / or rivaroxaban, with last dose of

- Ticagrelor < 72 hrs.
- Rivaroxaban < 48 hrs.




### Timing

Start therapy with the start of CPB. CytoSorb® is easily integrated into the CPB circuit (post-pump to venous reservoir)



### Dosing

Postoperative continuation (with a new adsorber integrated into an extracorporeal circuit) is normally NOT needed if CPB time > 60 min, but can be done if needed.

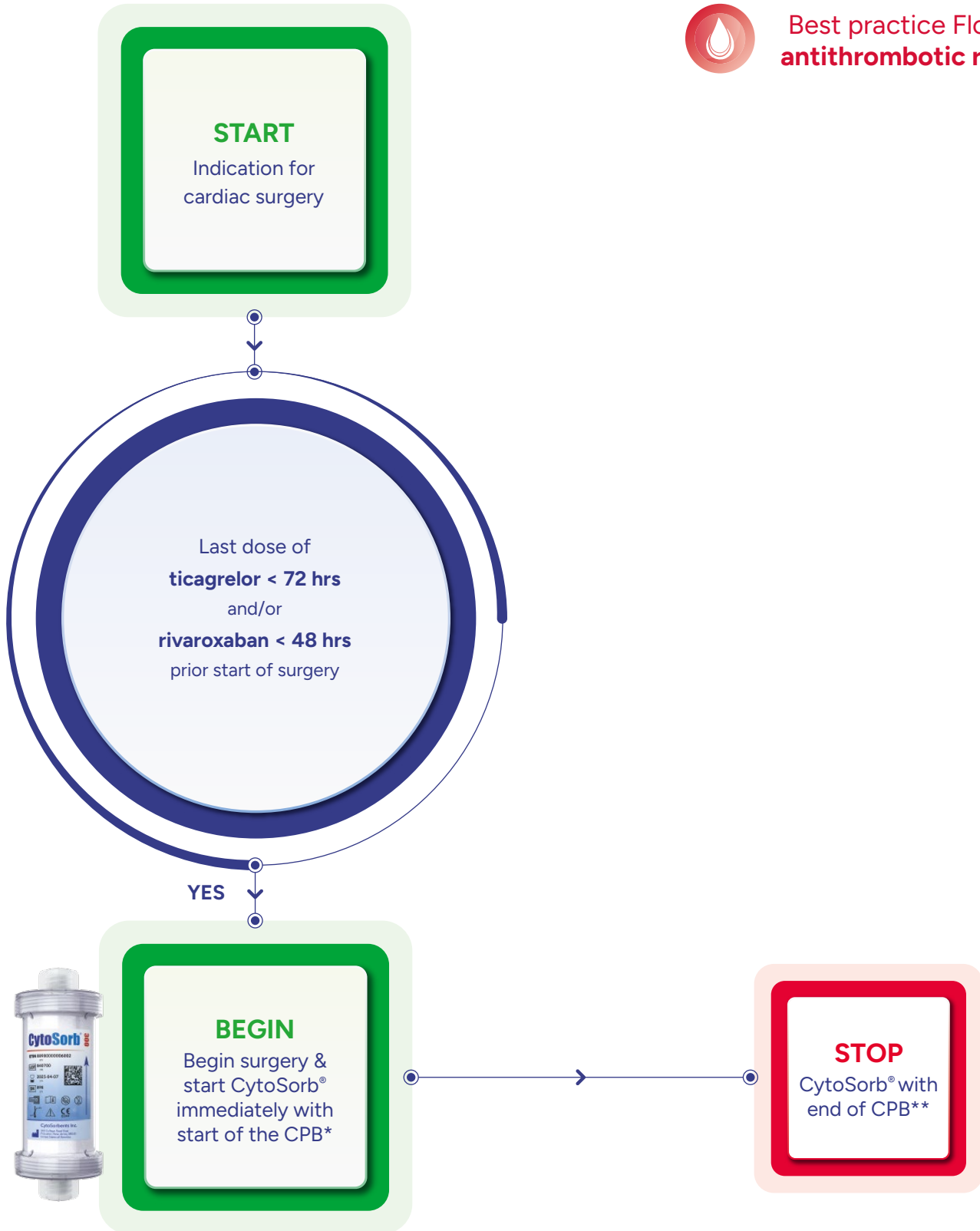

\* CytoSorb® integration into CPB

\*\* CPB times should ideally be >60 minutes to allow enough time for sufficient substance removal by CytoSorb®.



Learn more in detail  
[cyto.zone/setup](https://cyto.zone/setup)